


concret


ceiling

floor


Customers provide distance

Customers provide distance

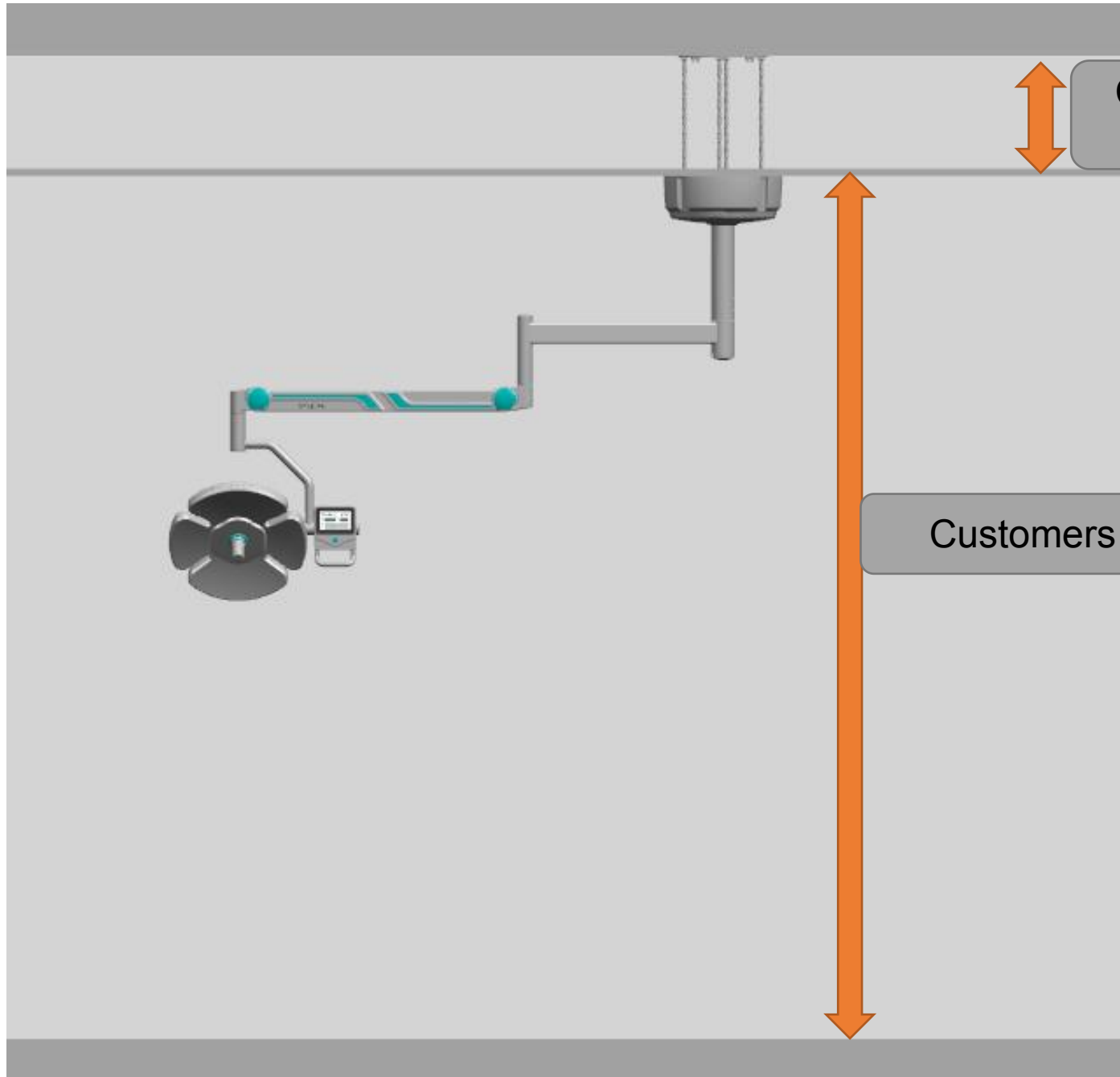

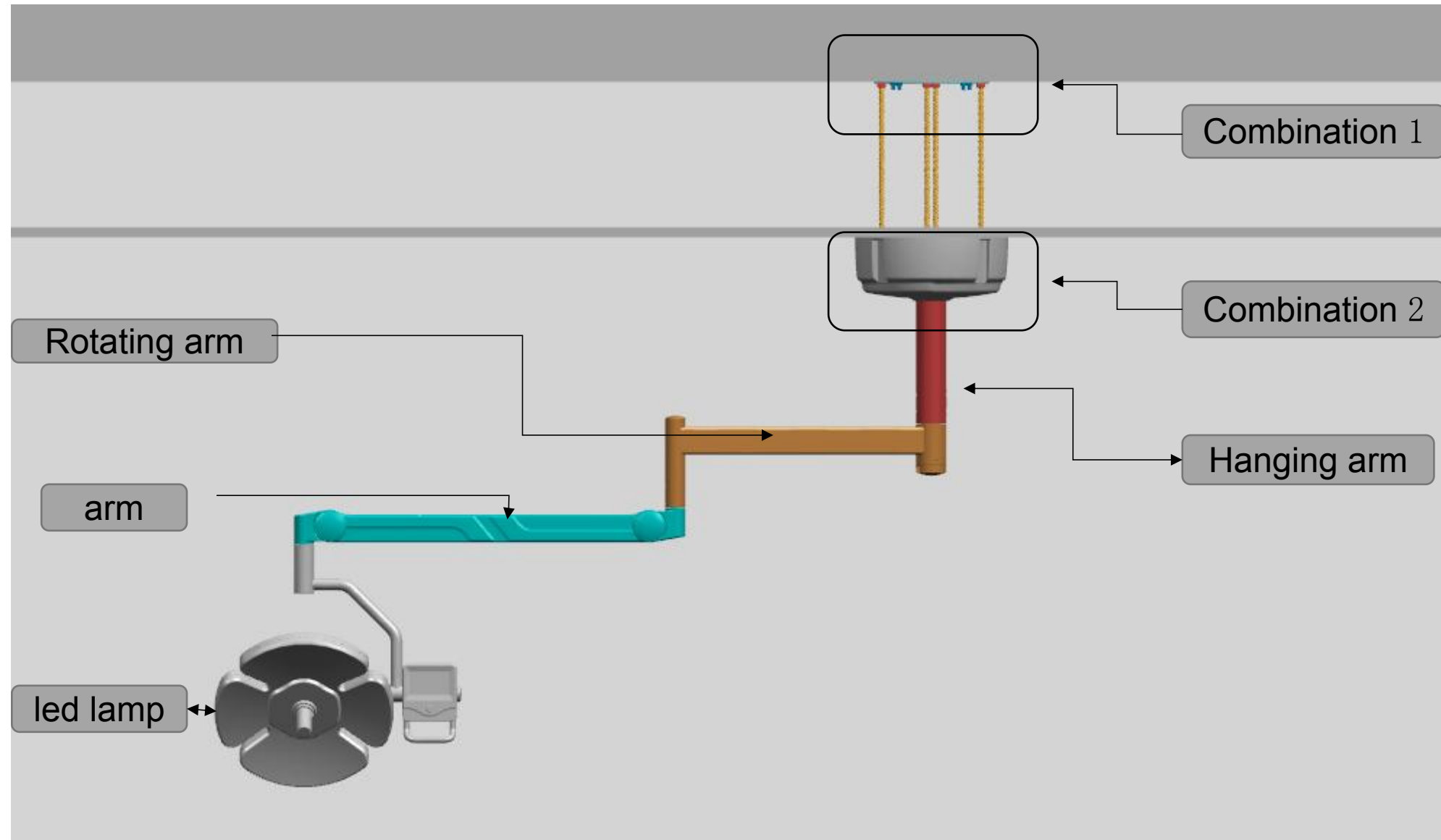


Combination 1

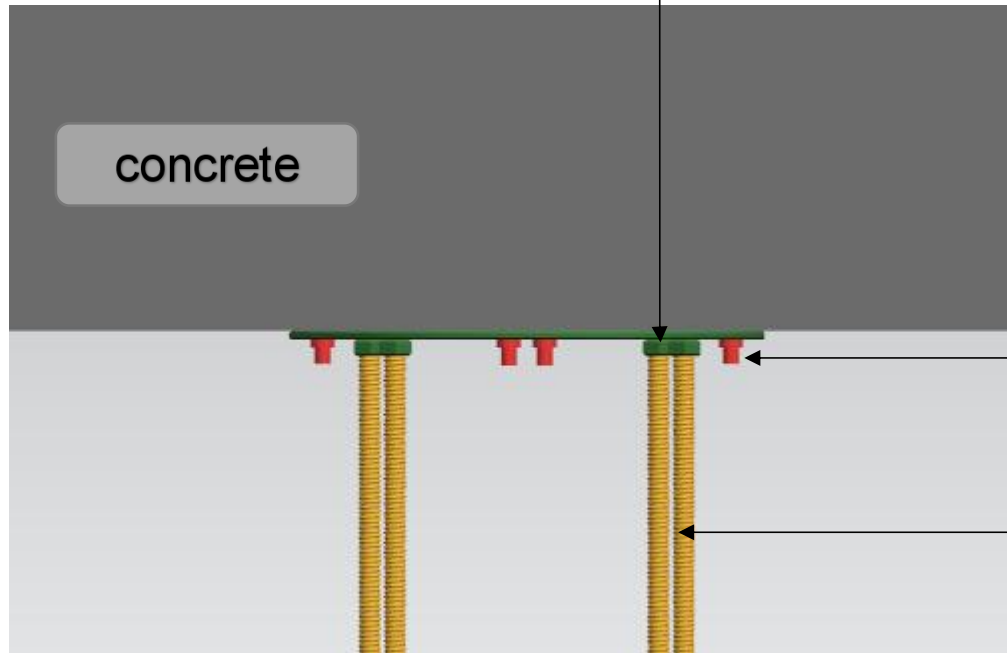

1: Pulls explodes the screws into the concrete



M10 expansion screws

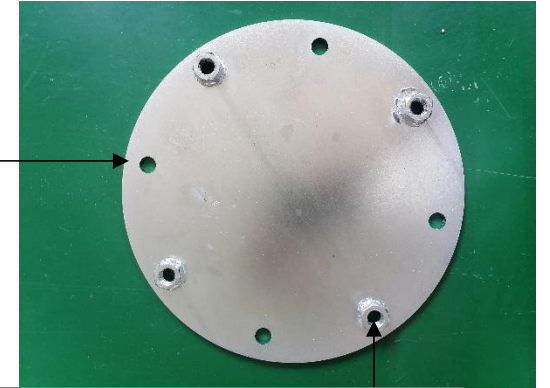

290mm bottom board

2: Install steeve



M14 steeve




Combination 2

DentalDeal

Electric power


Put steeve into holes

Put wires into hole

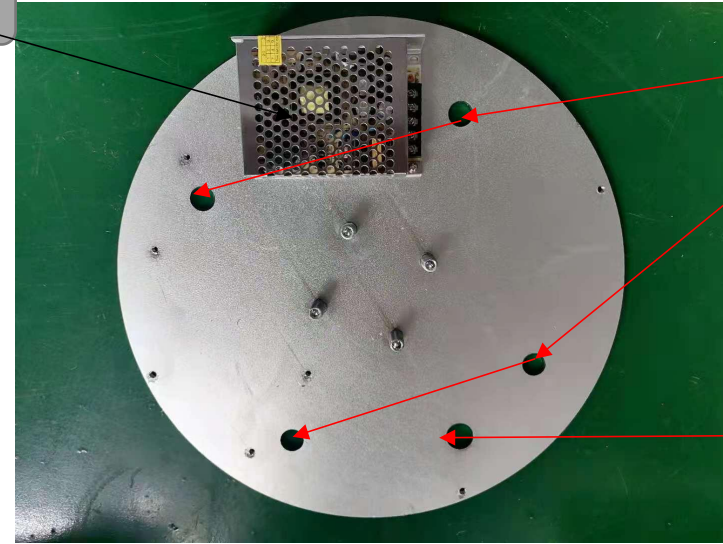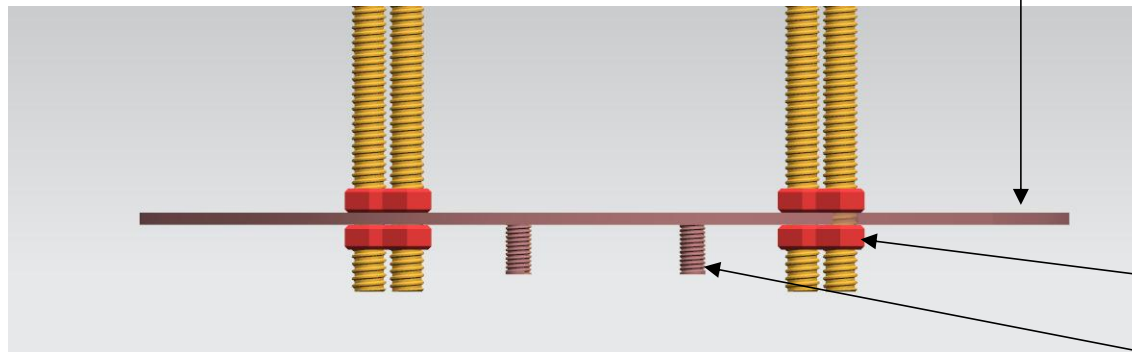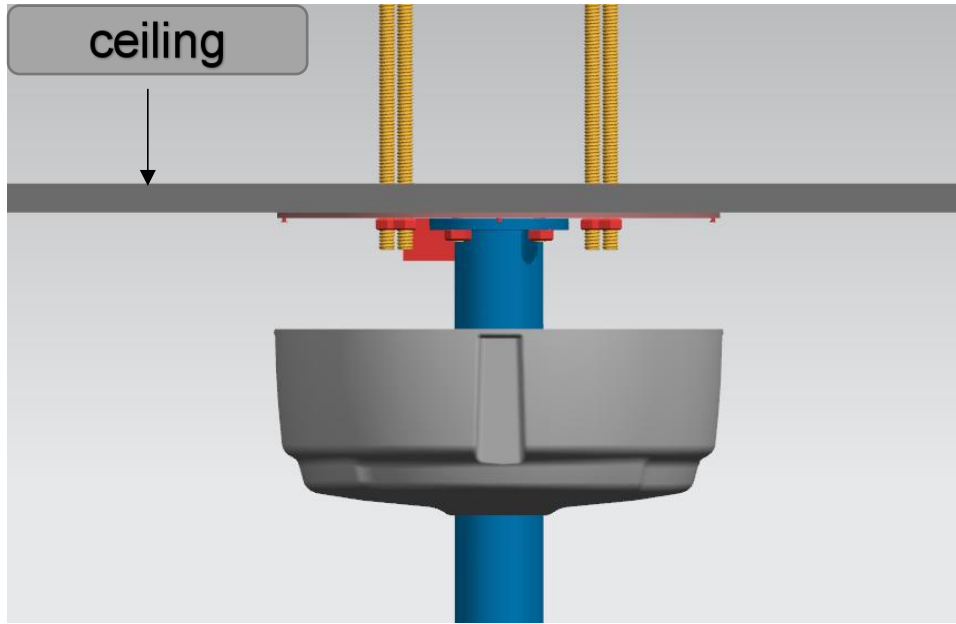

1: Make holes on the ceiling according to relevant positions


2: Use the strews of steeve to fix the disk, put the disk down , and then fix the screws

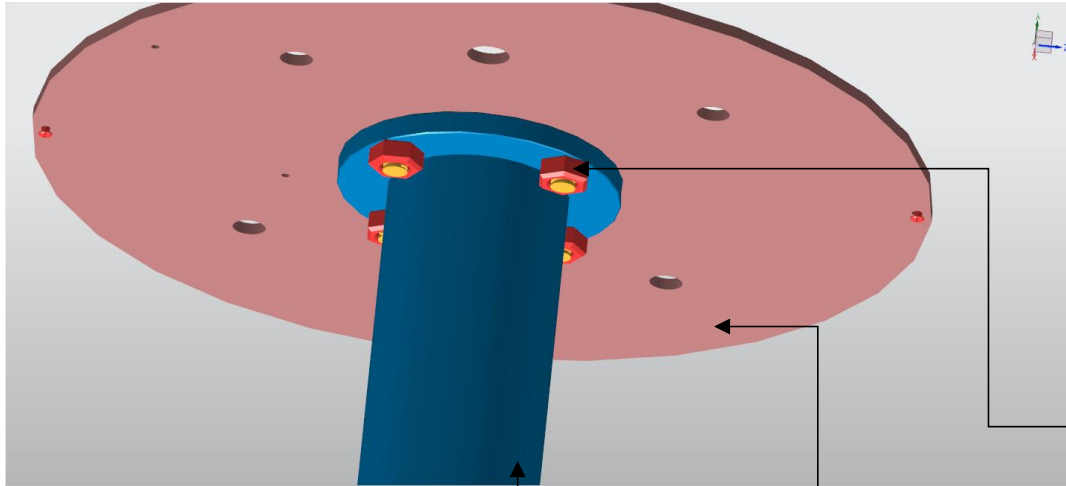


M10 screws



3: Use the screws to fix the hanging arm, and connect the power

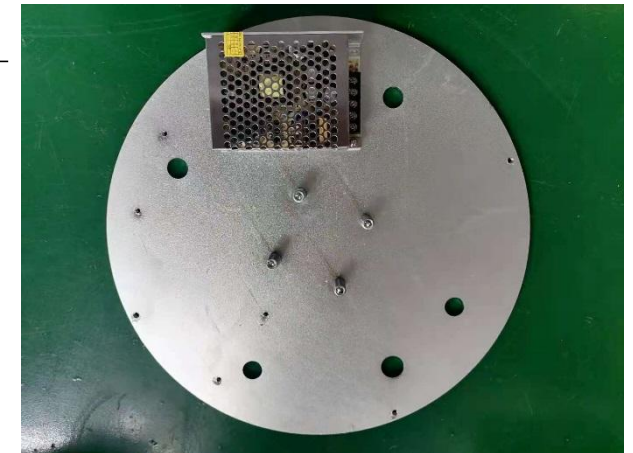


bottom board with power


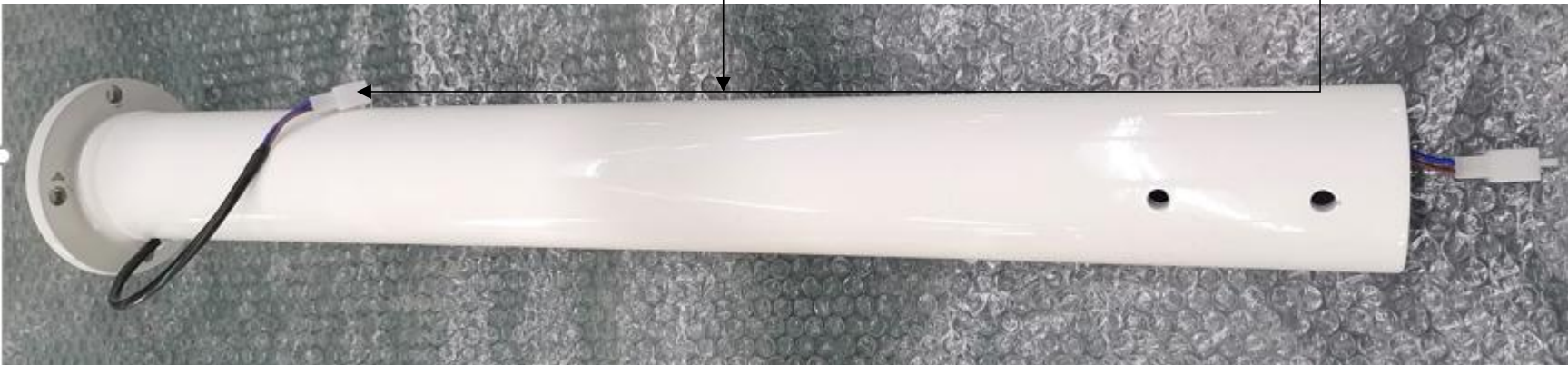



the length of hanging arm is customized accoring to the floor height

4: Use the screws to fix the disk

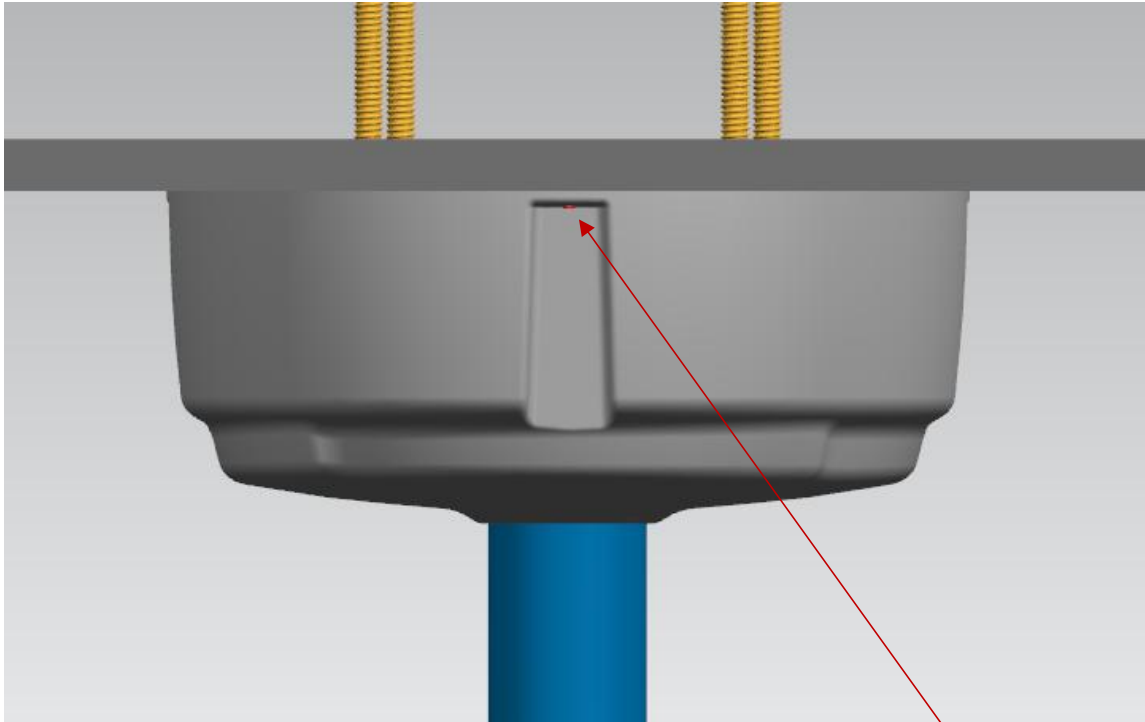

The fixed position
for 4 screws

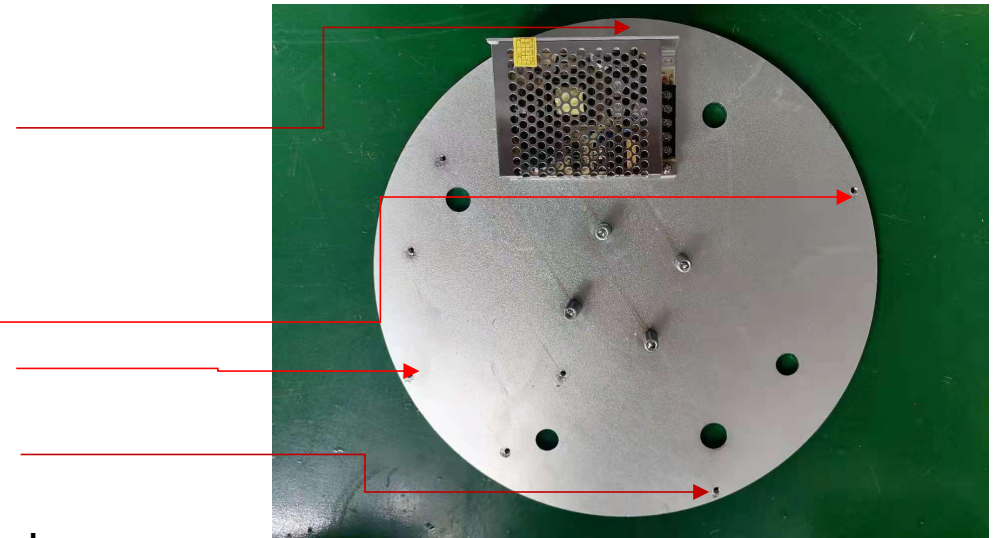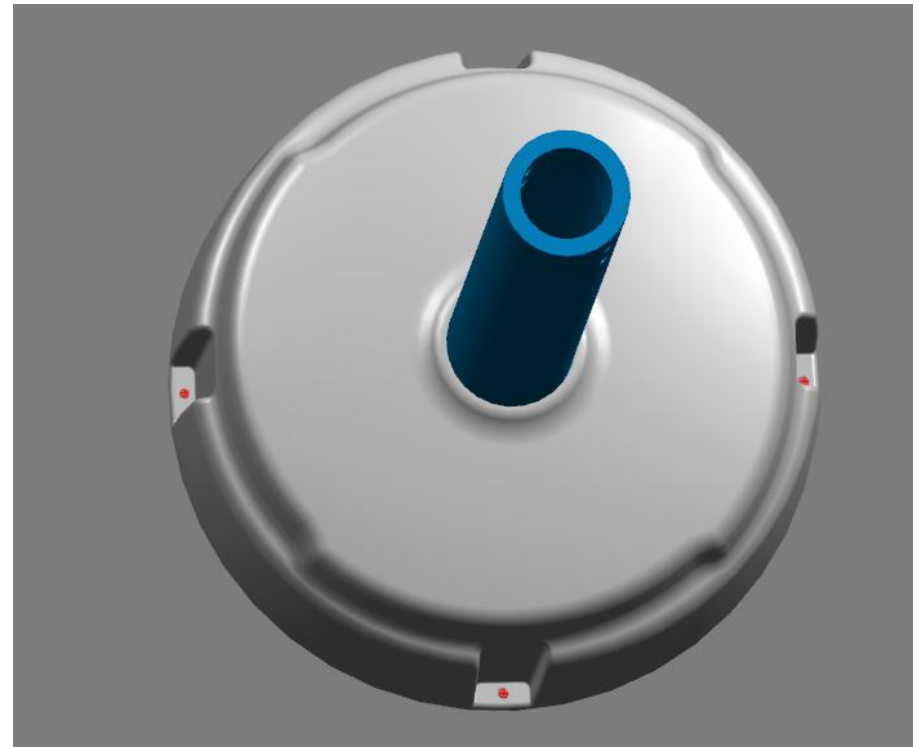

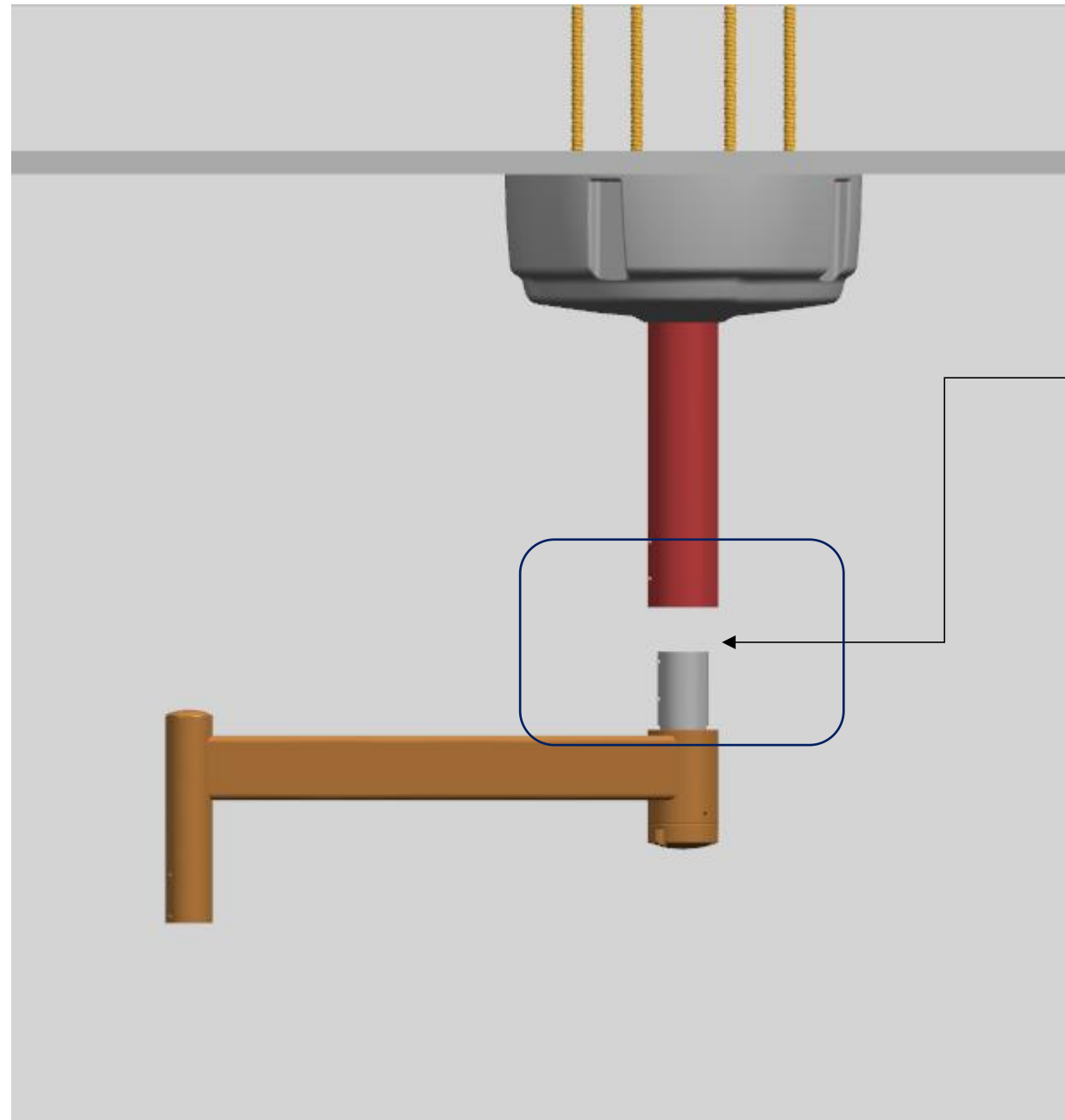

1: power line connection, connecting rotating arm with hanging arm, and then fix the 4 screws

